

# Washing Solution (Concentrate)

Read this package insert carefully before use. Keep it on hand for reference as needed.

If you have had or could have had any serious incident related to the device, please report it directly to the manufacturer or through the authorised representative and to your local regulatory authority.

## Intended Purpose

- ✦ Washing Solution (Concentrate) is a washing solution for use with OSMO STATION OM-6060 or OSMO STATION 2 OM-6070 which is intended for the quantitative measurement of osmolality. For *in vitro* diagnostic use and professional use only.

## Composition

- Sodium phosphate Tribasic, 0.2% (CAS No. 7601-54-9)
- Polyoxyethylene alkyl (12-14) ethers, 1% (CAS No. 84133-50-6)

## Usage

Dilute the concentrated solution with deionized water at a ratio of 1:9 for use (1 part concentrated solution diluted with 9 parts deionized water).

Read the operating manual of the relevant device carefully before use.

## Caution

1. Always use freshly made washing solution as fungus or bacteria may grow after it is diluted.
2. Use the washing solution within expiration date. If the storage is not appropriate, sufficient performances are not maintained even though it is within the expiration date.
3. Avoid direct sunlight for storage.
4. In case of contact with eyes, immediately and thoroughly wash with tap water. See a physician if necessary.
5. This product is alkaline washing solution. Do not drink. Wear protective gloves for safety before use.



### Disposal of Biohazardous Waste

Dispose of liquid waste according to your regulations on biohazardous waste.

## Storage and Stability

1. Storage  
Store at room temperature (1 - 30°C / 34 - 86°F).
2. Expiration date  
Printed on the bottle label and the outer container.

## Materials Provided

Washing Solution (Concentrate) 1 L × 1 bottle



### ARKRAY Factory, Inc.

1480 Koji, Konan-cho, Koka-shi  
Shiga 520-3306, JAPAN  
[https://www.arkray.co.jp/script/mailform/afc-contact\\_eng](https://www.arkray.co.jp/script/mailform/afc-contact_eng)



### ARKRAY Europe, B.V.

Prof. J.H. Bavincklaan 2  
1183 AT Amstelveen, THE NETHERLANDS

If you need to obtain technical assistance,  
please contact ARKRAY Europe, B.V.

TEL: +31-20-545-24-50  
FAX: +31-20-545-24-59



### UK RESPONSIBLE PERSON

#### ARKRAY Ltd.

Beacon House Stokenchurch Business Park  
Ibstone Road, Stokenchurch, High Wycombe  
HP14 3FE, UK  
<http://Arkray.co.uk>

