

Centrifuge Tube (for SPOTCHEM EZ)

Read the operating manual for the instrument before use. If you have had or could have had any serious incident related to the device, please report it directly to the manufacturer or through the authorised representative and to your local regulatory authority.

Intended Purpose

Centrifuge Tube (for SPOTCHEM EZ) is a dedicated whole blood sample container for use with SPOTCHEM EZ (SP-4430), which is intended for the quantitative measurement of several physiological markers. For *in vitro* diagnostic use and professional use only.

Warnings and Precautions


- For *in vitro* diagnostic use only.
- Take extra care when handling clinical samples and analytical tools. Dispose of biohazardous waste properly. For detailed instructions, see the operating manual for the instrument.
- Do not reuse.

Product Description

This sample tube contains heparin as an anticoagulant and is used to separate plasma from whole blood.

Directions

1. Pour 250 μ L of the freshly collected whole blood into the sample tube.
2. Recap tightly and invert gently 5 to 6 times to mix.
3. Remove the cap and immediately set the sample tube in the instrument's internal centrifuge for measurement.

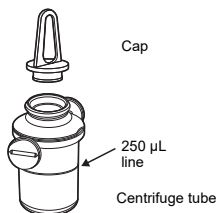

Cautions

1. Wear appropriate protective gear such as disposable gloves before handling samples.
2. Gently open the cap, taking care not to scatter the crystals coating the inner surface of the sample tube. They are lithium heparin used as an anticoagulant.
3. Do not use sample tubes that are not capped. Do not use the sample tube if the cap or centrifuge tube is damaged.
4. Store in a dark place avoiding humidity and direct sunlight.
5. The sample tube is not sterilized.
6. The sample tube is not intended for the storage of specimens.
7. Do not use this sample tube to measure whole blood collected in sample tubes containing anticoagulants other than heparin.

Storage and Stability

Store at 2 - 30°C (36 - 86°F)

Sample tubes may be used until the expiration date listed on the package.

Materials Provided

The following items are provided in the following quantities.

- Centrifuge Tube (for SPOTCHEM EZ): 100 pcs
- 100 centrifuge tubes
- 100 caps

Materials Required but Not Provided

Measurement also requires other consumables. For details, see the operating manual for the instrument.



ARKRAY Factory, Inc.

1480 Koji, Konan-cho, Koka-shi

Shiga 520-3306, JAPAN

https://www.arkray.co.jp/script/mailform/afc-contact_eng



ARKRAY Europe, B.V.

Prof. J.H. Bavinklaan 2

1183 AT Amstelveen, THE NETHERLANDS

If you need to obtain technical assistance,

please contact ARKRAY Europe, B.V.

TEL: +31-20-545-24-50

FAX: +31-20-545-24-59



UK RESPONSIBLE PERSON

ARKRAY Ltd.

Beacon House Stokenchurch Business Park

Ibstone Road, Stokenchurch, High Wycombe

HP14 3FE, UK

<http://Arkray.co.uk>

(84-09447A)

