



thinka Rapid Test series

CHW



ARKRAY provides
rapid and high accuracy
Test Kit series

thinka Canine Heartworm Ag Test kit CHW



Perform the Canine Heartworm test simple and quick in one step.

Simple : Only drop the sample.
Easy measurement in 1 step.


Quick : Within 10 minutes, you can get the result.

High accuracy : Easy to determine the test line with high sensitivity.

Performing a test



Drop a sample (2 drops) with pipette included.

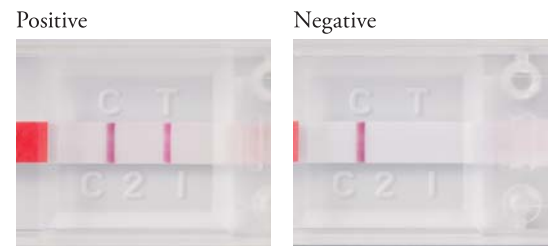

Determine the result

Clinical evaluation

Correlation with Other product A

		product A	
		Positive	Negative
This product	Positive	15	0
	Negative	0	17

positive concordance rate(%) = $15 / 15 \times 100 = 100\%$

negative concordance rate(%) = $17 / 17 \times 100 = 100\%$

Correlation with Other product B

		product B	
		Positive	Negative
This product	Positive	15	0
	Negative	0	17

positive concordance rate(%) = $15 / 15 \times 100 = 100\%$

negative concordance rate(%) = $17 / 17 \times 100 = 100\%$

Specification

Objective animals	Dog
Sample	Whole blood, blood serum, blood plasma
Measurement principle	Immunochromatography
Reaction time	5-10 minutes
Sample volume	Approx.80μL
Storage conditions	2 - 30°C
Expiration date	24 months
Package contain	Test cartridge (10 tests) Disposable Pipette (10 pcs)

ARKRAY GLOBAL BUSINESS, INC.

Kyoto Miyuki Bldg. 10F, 689 Takanna-cho,
Nakagyo-ku, Kyoto 604-8153, JAPAN

TEL +81-75-662-8967 FAX +81-75-662-8973